TRACOE medical GmbH  
 Reichelsheimer Straße 1 / 3, 55268 Nieder-Olm / Germany  
 Tel.: +49 6136 9169-0, Fax: +49 6136 9169-200  
 info@tracoe.com, www.tracoe.com

**Instructions for Use for TRACOE® *mini*  
 Tracheostomy Tubes  
 for Neonates (REF 350) and Children (REF 355)**

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**TRACOE**<sup>®</sup>  
*mini*

**ONLY**

Federal (USA) law restricts this device to the sale by or on the order of a physician

## Instructions for Use for TRACOE<sup>®</sup> *mini* Tracheostomy Tubes for Neonates (REF 350) and Children (REF 355)

### 1. General Information

**CAUTION:** Read these instructions carefully prior to use. These instructions are applicable only to the products mentioned above. All persons responsible for the use of these products and the care of patients must be familiar with these instructions.

**Sterilization:** The tubes are sterilized with ethyleneoxide. Contents are sterile unless the seal is broken or the package is opened or damaged.

**Single-Patient Product:** TRACOE *mini* tracheostomy tubes are medical products intended for only one patient. Multiple use on one and the same patient is allowed.

**Useful Life:** The manufacturer recommends using TRACOE *mini* tracheostomy tubes for a period not longer than 14 days. Tubes that show even the slightest damages must absolutely be replaced.

**Package Contents:** Each package contains a tube, an obturator, a neck strap, an instruction for use and a tube information card.

**Sizes/Dimensions:** Sizes and dimensions of the tubes are listed in table 1.

**Tube Information Card:** Each TRACOE *mini* tracheostomy tube will be delivered with a tube info card including two detachable labels. These labels show all product details (REF no., size, length, lot number). The tube info card should be kept separately. It makes reordering easier and should be enclosed in any complaint. The detachable label may be applied, for example, to the patient's record.

**Storage:** TRACOE *mini* tracheostomy tube must not be stored at a temperature of over 50 °C and under -20 °C.

**Components/Disposal:** TRACOE *mini* tracheostomy tubes for neonates and children consist of plastic materials that are internationally authorized for medical applications; they do not contain any latex or plasticizers including phthalates. The products must only be disposed in accordance with the national regulations.

## 2. General Description

TRACOE *mini* tracheostomy tubes for neonates and children consist of a soft and flexible neck flange with two wings and a 15 mm connector as well as a thin-walled, radiopaque, specially curved cannula. Each tube will be delivered with an obturator.

In addition to the TRACOE *mini* tubes of standard lengths, tubes for children (REF 355) are also available in special lengths according to customer's specifications.

### Accessories:

The TRACOE *mini* Extension Piece (REF 356) can be placed on all TRACOE *mini* tubes. It is particularly indicated for children with a very short neck in order to prevent the opening of the tube from dislocating.

The TRACOE humid assist mini (REF 643) is a heat / moisture exchanger for all children's tubes with a 15-mm connector and may be used for children with a tidal volume between 7 ml and 50 ml.

## 3. Indications

The tubes are intended for use when an access to the respiratory tracts through a tracheostoma is required.

#### 4. Instructions

- According to the anatomy of the patient, choose the tube of the respectively largest diameter possible (see table);
- Create access to the trachea using a standard tracheostomy technique;
- Take the tube and the obturator out of the package; insert the obturator into the tube;
- A thin layer of water-soluble lubricant may be applied to the tube and the protruding end of the obturator;
- Insert the tube with the obturator into the trachea of the patient and remove the obturator immediately afterwards;
- Ensure that the patient is ventilated properly through the tracheostomy tube;
- Secure the tube with a neck strap; once properly adjusted, it should be possible to place one finger between the patient's neck and the neck strap;
- If suctioning of secretions is required, it should be carried out very carefully.

**CAUTION:** Although the 15 mm connector of TRACOE *mini* tubes fully complies with international standards, it must be generally ensured when attaching a female connector counterpart, that it can be removed without any difficulties. This applies to speaking valves, heat and moisture exchangers (e.g. HMEs) or respirators likewise. When using speaking valves or HMEs in combination with a TRACOE *mini* tube, the patient must be monitored permanently; the same applies to mechanical ventilation. When using respirators, it must be ensured that an appropriate ventilation mode, taking account of the inner diameter of the cannula, is set at the respirator (e.g. pressure-supporting ventilation without control of the ventilation volume).

**CAUTION:** Contact between electro-surgical electrodes or a laser and the tubes must be avoided; such contact, especially in the presence of gas mixtures with a high concentration of oxygen, could result in combustion of the tube and the release of toxic combustion products, including hydrochloric acid (HCL).

#### **CAUTION: Damage Resulting from the Use of Force**

Do not use force on the tracheostomy tube, because otherwise there is the risk of damage (e.g., breaking). In case of firmly attached connections that can only be removed with difficulty, a separation tool approved for tracheostomy tubes must be used.

## 5. Care, Cleaning and Disinfection

The care of TRACOE *mini* tubes is based on regular, thorough cleaning. The tubes can be cleaned with normal household means like for example mild detergents. For cleaning TRACOE *mini* tubes we recommend the use of a complete TRACOE tube clean cleaning set (REF 930-A or – B with cleaning powder or REF 931-A or –B with concentrated cleaning liquid). The sets include a cleaning tub with a mesh insert, cleaning agent, cleaning swabs, and cleaning brush. When cleaning the outer cannula, care must be taken to ensure that the cuff is not damaged. The outer and inner cannulas are placed separately in the mesh insert. This will then be immersed in the cleaning solution in the tub. The best way of removing any secretion deposits inside the cannulas is by using the small or large TRACOE tube clean cleaning swabs (REF 935 and REF 936). The cleaning swabs consist of a flexible plastic stick with a fluff-free polyurethane sponge attached to the end. This avoids introducing fluff into the cannula or trachea. For impurities that are difficult to remove, a cleaning brush with a rounded tip and soft bristles should be used (TRACOE cleaning brush REF 938 or REF 940). On no account should brushes with a sharp edged end be used as these can scratch the soft material of the cannulas. After cleaning, rinse the cannulas with tap water or distilled water and let them dry. For further details, see Instructions for Use for TRACOE tube clean cleaning sets.

**CAUTION: The products must never be cleaned with aggressive household detergents, denture cleansers, or concentrated alcohol as these may have a lasting adverse effect on the function of the product.**

**Disinfection:** Since the upper respiratory tract is never without germs – even in healthy persons – we do not recommend the use of disinfection.

If, however, disinfection appears necessary, TRACOE medical can provide, on request, a list of tested disinfectants. This list can also be accessed on the website of TRACOE medical GmbH ([www.tracoe.com](http://www.tracoe.com)).

**CAUTION:** Under no circumstances disinfectants may be used that release chlorine or substances that contain strong alkalis or phenol derivatives.

**CAUTION:** Before re-using or re-inserting a cannula, it must be examined to ensure that it is intact and functions correctly. Checks should

be performed for absence of obstructions, stable connection between the cannula and the neck flange, intact patient end etc.

**Recommendation:** It is urgently recommended that a replacement tube is always available. This must be stored in a clean and dry condition.

## **6. Returns**

Returns of used products are accepted only if accompanied by a completed decontamination certificate. You can obtain the required form either directly from TRACOE medical or from the website ([www.tracoe.com](http://www.tracoe.com)).

## **7. General Terms and Conditions**

The sale, delivery and return of all TRACOE products are carried out exclusively according to the General Terms and Conditions which can be obtained directly from TRACOE medical GmbH or from the website ([www.tracoe.com](http://www.tracoe.com)).

TRACOE medical GmbH  
Reichelsheimer Straße 1 / 3, 55268 Nieder-Olm / Germany  
Tel.: +49 6136 9169-0, Fax: +49 6136 9169-200  
info@tracoe.com, www.tracoe.com

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**Useful Life:** The manufacturer recommends using TRACOE *mini* tracheostomy tubes for a period not longer than 29 days.\* Tubes that show even the slightest damages must absolutely be replaced.

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## 4. Instructions

- According to the anatomy of the patient, choose the tube of the respectively largest diameter possible (see table);
- Create access to the trachea using a standard tracheostomy technique;

- Take the tube and the obturator out of the package; insert the obturator into the tube;
- A thin layer of water-soluble lubricant may be applied to the tube and the protruding end of the obturator;
- Insert the tube with the obturator into the trachea of the patient and remove the obturator immediately afterwards;
- Ensure that the patient is ventilated properly through the tracheostomy tube;
- Secure the tube with a neck strap; once properly adjusted, it should be possible to place one finger between the patient's neck and the neck strap;
- If suctioning of secretions is required, it should be carried out very carefully.

**CAUTION:** Although the 15 mm connector of TRACOE *mini* tubes fully complies with international standards, it must be generally ensured when attaching a female connector counterpart, that it can be removed without any difficulties. This applies to speaking valves, heat and moisture exchangers (e.g. HMEs) or respirators likewise. When using speaking valves or HMEs in combination with a TRACOE® *mini* tube, the patient must be monitored permanently; the same applies to mechanical ventilation. When using respirators, it must be ensured that an appropriate ventilation mode, taking account of the inner diameter of the cannula, is set at the respirator (e.g. pressure-supporting ventilation without control of the ventilation volume).

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**CAUTION: Damage Resulting from the Use of Force**

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## 5. Care, Cleaning and Disinfection

The care of TRACOE® *mini* tubes is based on regular, thorough cleaning. The tubes can be cleaned with normal household means like for example mild detergents. For cleaning TRACOE® *mini* tubes we recommend the use of a complete TRACOE® tube clean cleaning

set (REF 930-A or – B with cleaning powder or REF 931-A or –B with concentrated cleaning liquid). The sets include a cleaning tub with a mesh insert, cleaning agent, cleaning swabs, and cleaning brush. When cleaning the outer cannula, care must be taken to ensure that the cuff is not damaged. The outer and inner cannulas are placed separately in the mesh insert. This will then be immersed in the cleaning solution in the tub. The best way of removing any secretion deposits inside the cannulas is by using the small or large TRACOE® tube clean cleaning swabs (REF 935 and REF 936). The cleaning swabs consist of a flexible plastic stick with a fluff-free polyurethane sponge attached to the end. This avoids introducing fluff into the cannula or trachea. For impurities that are difficult to remove, a cleaning brush with a rounded tip and soft bristles should be used (TRACOE® cleaning brush REF 938 or REF 940). On no account should brushes with a sharp edged end be used as these can scratch the soft material of the cannulas. After cleaning, rinse the cannulas with tap water or distilled water and let them dry. For further details, see Instructions for Use for TRACOE® tube clean cleaning sets.

**CAUTION:** The products must never be cleaned with aggressive household detergents, denture cleansers, or concentrated alcohol as these may have a lasting adverse effect on the function of the product.

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**CAUTION:** Before re-using or re-inserting a cannula, it must be examined to ensure that it is intact and functions correctly. Checks should be performed for absence of obstructions, stable connection between the cannula and the neck flange, intact patient end etc.

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## **7. General Terms and Conditions**

The sale, delivery and return of all TRACOE® products are carried out exclusively according to the General Terms and Conditions which can be obtained directly from TRACOE medical GmbH or from the website ([www.tracoe.com](http://www.tracoe.com)).

TRACOE medical GmbH  
Reichelsheimer Straße 1 / 3, 55268 Nieder-Olm / Germany  
Tel.: +49 6136 9169-0, Fax: +49 6136 9169-200  
info@tracoe.com, www.tracoe.com

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