PROVOX
Life

Comfort Adhesive

Provox Life Comfort Adhesive

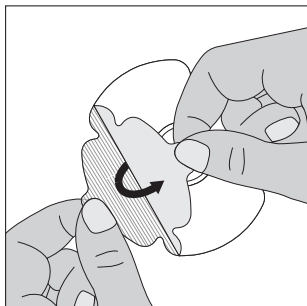
Figure 1

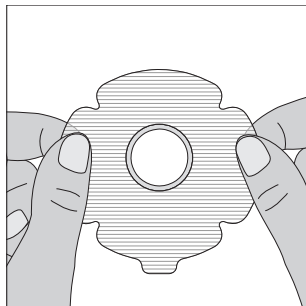
Figure 2

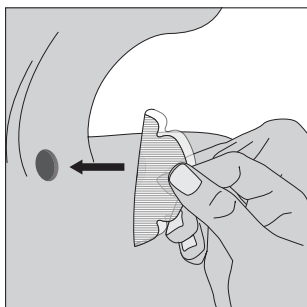
Figure 3

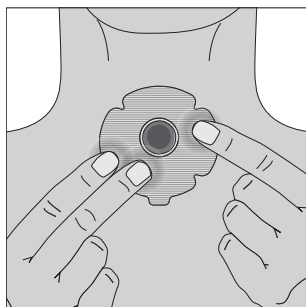
Figure 4

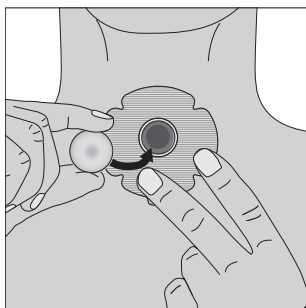
Figure 5

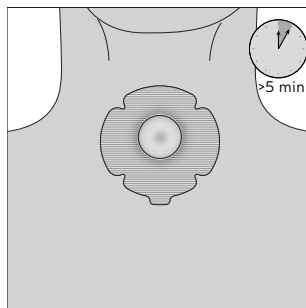
Figure 6

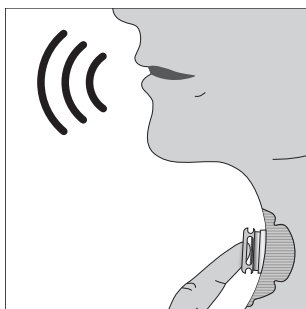
Figure 7

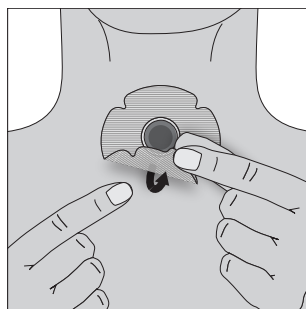
Figure 8

Disclaimer

Atos Medical offers no warranty—neither expressed nor implied—to the purchaser here under as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Compatible Products; Kompatible Produkte; Compatibele producten; Produits compatibles; Prodotti compatibili; Productos compatibles; Produtos compatíveis; Kompatibla produkter; Kompatible produkter; Kompatible produkter; Yhteensopivat tuotteet; Samhæfar vörur; Ühilduvad tooted; Kompatibilní produkty; Kompatibilis termékek; Kompatybilne produkty; Kompatibilni proizvodi; Συμβατά προϊόντα; 対応製品; المنتجات المتوافقة

Provox Life Comfort Adhesive

Provox Life HMEs
Provox Life Oxygen
Provox Life Shower
Provox Life Baseplate Adaptor
Provox Life LaryTube Standard with Ring
Provox Life LaryTube Fenestrated with Ring
Provox Cleaning Towel

Contents

EN - ENGLISH	7
DE - DEUTSCH	10
NL - NEDERLANDS	13
FR - FRANÇAIS	16
IT - ITALIANO	19
ES - ESPAÑOL	22
PT - PORTUGUÊS	25
SV - SVENSKA	28
DA - DANSK	31
NO - NORSK	34
FI - SUOMI	37
IS - ÍSLENSKA	40
ET - EESTI	43
CS - ČESKY	46
HU - MAGYAR	49
PL - POLSKI	52
HR - HRVATSKI	55
EL - ΕΛΛΗΝΙΚΑ	58
AR - العربية	63

Provox Life System

Provox Life devices are designed to work together as a system. Provox Life system consists of Provox Life Adhesives, Provox Life HMEs, Provox Life LaryTubes, Provox Life LaryButtons, and a range of Provox Life accessories.

The devices should only be used in accordance with the Instructions for Use (IFU).

Provox Life Comfort Adhesive

1. Intended Use

Provox Life Comfort Adhesive is a single-use adhesive that provides attachment for Provox Life HMEs and accessories after total laryngectomy.

2. Indications

Facilitation of pulmonary rehabilitation after total laryngectomy. The product is intended for adults (18+ years).

3. Contraindications

Patients without the physical, cognitive, or mental ability required to attach, remove, or operate the devices themselves, should not use the devices independently and should only use them if they are under sufficient supervision of a clinician or a trained caregiver. If not, this may lead to restricted breathing and/or suffocation.

4. Description of the Device

Provox Life Comfort Adhesive is made of a silicone gel material that is suitable for sensitive and irritated skin. Dependent on clinical suitability and continued tolerance, the product can be used postoperatively and during and after radiotherapy.


Provox Life Comfort Adhesive has been evaluated to be safe to use during radiotherapy (EBRT and IMRT at a dose of 2-3 Gy per radiation session).

Provox Life Comfort Adhesive is for single use only and shall be changed at least every 24 h or when clinically indicated, e.g. soiled or loose.

5. Warnings

- The skin in contact with the adhesive needs to be assessed both on an individual basis and reviewed regularly for microbial growth that could potentially lead to infection.
- Reuse may cause transfer of microorganisms leading to infection.

Provox Life Comfort Adhesive

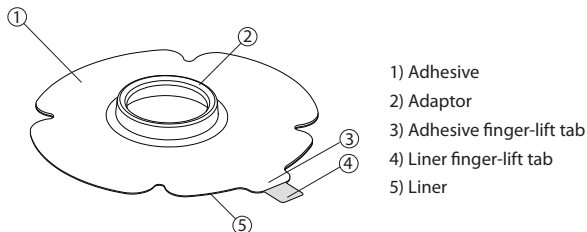6. Precautions

- The adhesive may irritate the skin. Stop using the adhesive if skin irritation develops and consult your clinician.
- Only use Provox Life Comfort Adhesive with compatible HMEs and accessories. When taking a shower using a Provox Life Comfort Adhesive, use Provox Life Shower to prevent water entering the lungs, which may lead to aspiration pneumonia and coughing (see the IFU accompanying Provox Life Shower).

Side-Effects

Possible side-effects related to the use of Provox Life Comfort Adhesive may include skin irritation, allergic skin reaction, and/or skin maceration.

7. How to Use

7.1 Clean

1. Clean the skin according to local guidelines, e.g. lukewarm water.
 2. Let the skin dry before applying the adhesive.
- Always clean mucus away from the adhesive, adaptor, and stoma after coughing.

7.2 Prepare and Apply

1. Remove the liner (see Figures 1–2).
2. Apply the adhesive to the skin around the stoma (see Figure 3).
3. Massage the adhesive gently (see Figure 4).
4. Attach a Provox Life HME (see Figure 5).
5. Wait at least 5 min before speaking to improve the adhesion (see Figures 6–7). Waiting longer may improve adhesion.

7.3 Remove

Provox Life Comfort Adhesive is for single use only and shall be changed at least every 24 h or when clinically indicated, e.g. soiled or loose.

1. Gently remove the adhesive using the finger-lift tab (see Figure 8).
2. Remove potential adhesive residue from the skin. While removing potential adhesive residue, prevent particles/fluids from entering the stoma.

Information

The products are intended to be stored in the box until removed for use, as the box protects the products and the label on the box contains important information such as batch code, use-by date, and a unique barcode.

8. Disposal

Always follow medical practice and national requirements regarding biohazards when disposing of a used medical device.

9. Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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Atos and the Atos logo are trademarks of Coloplast A/S.
© 2025 Coloplast A/S. All rights reserved.
Atos Medical UK Ltd Tottle Road
Atos Medical AB